
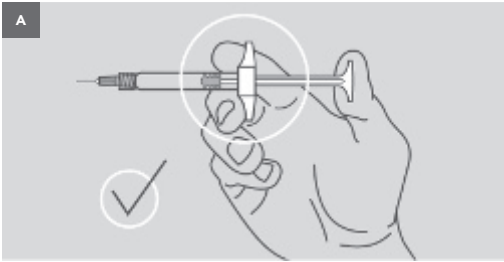
- F** NOTICE BELOTERO® HYDRO
- GB** INSTRUCTIONS FOR USE BELOTERO® HYDRO
- D** GEBRAUCHSINFORMATION FÜR BELOTERO® HYDRO
- I** ISTRUZIONI PER L'USO DI BELOTERO® HYDRO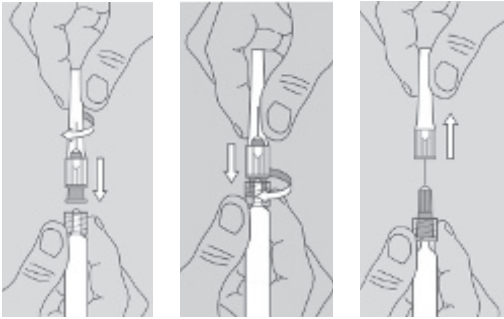

Ailette en position adéquate pour l'injection
 Backstop in the right position during injection
 Korrekte Position der Fingerauflage bei der Injektion.
 Posizione corretta del pistone durante l'iniezione

F NOTICE BELOTERO® HYDRO

Description

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GB INSTRUCTIONS FOR USE BELOTERO® HYDRO

Description

BELOTERO Hydro is a sterile, non-pyrogenic, viscoelastic, colourless, transparent gel of non-animal sodium hyaluronate, in a physiological phosphate buffer.

Presentation

BELOTERO Hydro is presented in a single use pre-filled glass syringe sterilized by moist heat. Each box contains one instruction leaflet, one syringe, two traceability labels and two sterile CE-marked needles for single use only. The dimensions of needles are stated on the external box.

BELOTERO Hydro has been developed to be used with 30G to 32G needles. To ensure optimal use of BELOTERO Hydro, it is recommended to use the needles provided in the box.

Composition

Sodium hyaluronate: 18.0 mg/ml
Glycerol: 21.0 mg/ml
Phosphate-citrate buffer pH 7 q.s.: gel volume

The volume of the gel in each syringe is stated on the external box.

Indications

BELOTERO Hydro is an injectable resorbable gel indicated to rehydrate seriously dehydrated or wrinkled skin tissue. This gel enables increasing the firmness and elasticity of the skin, improving the skin's complexion and radiance, wrinkles prevention on face, neck, cleavage or the back of the hands.

Posology, administration method and duration of effects

BELOTERO Hydro is designed to be injected into the superficial dermis or the dermo-epidermal junction by a legally approved practitioner.

BELOTERO Hydro must be injected into a healthy, non-inflamed, disinfected skin.

To ensure the optimal use of BELOTERO Hydro it is recommended to assemble the needle according to the diagram below.

BELOTERO Hydro may be injected using the multipuncture or the micro-papular techniques. It is advisable to use the multipuncture technique over the whole face performing a double series of parallel injections, the 2 series being perpendicular to each other. The micro-papules obtained by repeated injections of small volumes can be considered on the most dehydrated areas. If this latter technique is used, it is recommended that the patient be informed that the micro-papules may be visible for 2 to 3 days.

BELOTERO Hydro may be used in combination with all BELOTERO fillers respecting the correct depth of injection and indication for each product. If the needle becomes obstructed and the injection pressure becomes too great, stop the injection and change the needle.

After the injection, the practitioner can massage gently the treated area in order to distribute the product uniformly.

Hyaluronic acid and glycerol are naturally occurring components of the skin and also responsible for water retention and balance in the tissues. Exogenous sodium hyaluronate and glycerol injected into the very superficial dermal tissue should increase the water retention capacity in dehydrated skin sodium hyaluronate and glycerol, as they are identical to naturally occurring components, undergo the same metabolic process. The turnover of natural hyaluronic acid and glycerol in the skin's tissue is a matter of only a few days at maximum. BELOTERO Hydro presents a high concentration of sodium hyaluronate and glycerol which should maintain the rehydrating effect for about 15-30 days, the product slowly degrading over this period. As the skin has recovered its physiological properties, the effects will last for a longer period depending on age and the condition of the patient's skin.

Depth of injection

For best performance of the product, BELOTERO Hydro should be injected into the superficial dermis or the dermo-epidermal junction.

Contra-indications

BELOTERO Hydro is contra-indicated:

- In cases of known hypersensitivity to one of the product's components, especially hyaluronic acid,
- In pregnant or breast-feeding women,
- In young people less than 18 years old.

Do not inject BELOTERO Hydro into blood vessels.

Precautions for use

Before treatment, the patient must be informed about the device, its contra-indications and possible side effects.

Do not use BELOTERO Hydro for any indication other than skin tissue rehydration.

In the absence of clinical data on tolerance and efficacy on the injection of BELOTERO Hydro in patients with antecedents or an active autoimmune disease, the practitioner must decide whether to inject BELOTERO Hydro on a case-by-case basis depending on the nature of the disease as well as the associated treatment. It is recommended to propose a prior double-test on these patients and to not inject if the disease is evolving. It is also recommended to carefully monitor these patients after injection.

It is recommended not to inject BELOTERO Hydro in patients with a history of streptococcal diseases or in patients pre-disposed to hypertrophic scars or cheloids before practitioner's evaluation.

Check the integrity of the inner packaging prior to use, and the expiry date for both the syringe and the needle. Do not use these products if the expiry date has lapsed or if the packaging has been opened or damaged.

Sensitive skin may be pre-treated using a local anaesthetic patch or cream. It should be noted that anaesthesia may cause local redness or hypersensitivity.

No clinical data on the use of BELOTERO Hydro in combination with any other aesthetic medicine treatment, other than those belonging to BELOTERO range, is available.

Do not transfer BELOTERO Hydro into another container and do not add other ingredients to the product. Only the gel is sterile but not the outside of the syringe.

This product should not be used with an automated injection system not recommended by MERZ/ANTEIS. If an automated system is used, the practitioner should be entitled to the use of such a system, and read the instructions for use of the system prior to use.

Use only the needles provided in the set.

BELOTERO Hydro is a single-use product.

Discard the syringe and remaining product after use. Do not re-sterilize and do not reuse.

The patient must avoid applying make-up for at least 12 hours after treatment, and avoid saunas and Turkish baths, and prolonged exposure to the sun and UV rays for 2 weeks after the treatment. Patients should also avoid putting pressure on and/or handling the area of the face which has been treated. When the micro-papular technique has been used, the patient can massage the papules at the condition that the hands and treated area are disinfected.

Incompatibilities

Sodium hyaluronate precipitates in the presence of quaternary ammonium compounds (such as benzalkonium chloride). It is therefore recommended that BELOTERO Hydro does not come into contact with such substances.

Side effects

Possible side effects exist and must be described to the patient before treatment. Very slight bleeding may occur during the injection although this disappears spontaneously as soon as the injection is finished. In occasional cases, one or more of the following may occur either immediately or as a delayed reaction (list not exhaustive):

Local oedema or slight erythema, sometimes associated with stinging in the treated area. These reactions may last for a week.

Allergy to any of the product's components, especially sodium hyaluronate. Redness or discoloration of the treated area.

The patient should be instructed to report any side effect which lasts for more than one week to his/her practitioner. The practitioner may then prescribe the patient appropriate treatment.

Assembly of needle to syringe

For safe use of BELOTERO Hydro, it is important that the needle be properly assembled. See diagrams 1, 2, 3.

1. After having removed the luer-lock's cap and the needle's seal, push and screw the needle in its shield onto the syringe's luer-lock until you feel some counter-pressure.
2. Take a firm grip on the wider part of the needle shield, press and turn it a quarter of a turn.
3. Pull off the needle shield.

Storage

Store between 2 °C and 25 °C. Protect from light and freezing. Avoid mechanical shocks.

References

Updated documentation may be available from ANTEIS SA, Switzerland.

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Becton Dickinson Medical (S) Pte Ltd,